# DENTAPEN

*by Juvaplus*

FR	Seringue électronique pour Anesthésie Dentaire	3
EN	Powered Injector for Dental Anesthesia	4
ES	Inyector motorizado para anestesia dental	5
DE	Elektrischer Injektor für Dentalanästhetika	6
IT	Iniettore Motorizzato per Anestesia Dentale	7
PT	Injetor alimentado para anestesia dentária	8
NL	Elektronische spuit voor tandheelkundige anesthesie	9
PL	Aparat do iniekcji znieczulenia stomatologicznego	10
EL	Τροφοδοτούμενος εγχυτήρας για οδοντιατρική αναισθησία	11
BG	Захранван инжектор за дентална анестезия	12
CS	Vstříkovací nástroj pro dentální anestezii s vlastním pohonem	13
DA	Injektor for Dental Anæstesi	14
ET	Elektriline sissepritse pihusti hambaravi anesteesia jaoks	15
FI	Hammashoidon puudutuslaite	16
HU	Elektromos meghajtású fecskendező fogászati érzéstelenítéshez	17
LT	Elektrinis injektorius dantų anestezijai	18
LV	Ar bateriju darbināms zobu anestēzijas dozators	19
RO	Injector electric pentru anestezie dentară	20
SK	Elektrický injektor na dentálnu anestéziu	21
SL	Napajalni vmesnik za zobozdravstveno anestezijo	22
SV	Injektor för dentalanestesi	23
HR	Mehanička štrcaljka za dentalnu anesteziju	24
NO	Elektronisk sprøyte for dental anestesi	25
	EMC Information	27

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- Led bleue (clignotement lent) : pression normale

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# EN Instruction for use

## DEVICES DESCRIPTION

DENTAPEN is a non-sterile, cordless, and motorized device, powered by a lithium battery, intended to assist medical practitioners to inject dental anesthesia.

DENTAPEN is to be used with:

- A cartridge holder, compatible with most 1.7/1.8ml/2.2ml anesthesia cartridges on the market and with standard dental needles (Ref. DP03, DP05 or DP07),
- Finger grip to connect DENTAPEN with the cartridge holder, and to activate DENTAPEN (Ref. DP02 or DP04),
- Single-use sleeves to protect DENTAPEN during the operation (Ref. PS02)
- O-ring to allow active aspiration (Ref. OR01),
- A battery Lithium 6V/170 mAh (certified IEC60086-4) (Ref. BP04).

The use of any other accessory may deteriorate the functioning of DENTAPEN.

## GENERAL RECOMMENDATIONS

DENTAPEN is intended for professional use only. Its use and the results thereof are the exclusive responsibility of the practitioner.

The practitioner shall proceed only after having fully familiarized with both the product and its instructions for use.

The practitioner is sole responsible for evaluating the indications and contraindications relative to the injection technique and the anesthesia to administer.

- Always check whether DENTAPEN is functioning properly before use.
- Check the proper aspect of the aspiration O-ring.

## PRECAUTIONS FOR USE

Do not sterilize DENTAPEN or the battery.

Any alteration of DENTAPEN may cause its dysfunctions.

Dispose of the cartridge holder, finger grips or battery according to your local environmental regulations.

Conform to EMC information (see last page).

Keep the DENTAPEN out of reach of children.

Keep DENTAPEN away from water and chemical products.

DENTAPEN should not be used in an inflammable environment

Battery may present a fire or chemical burn hazard if mistreated. Do not recharge battery, open, dispose of in fire, put in backwards, mix with used or other battery types, short circuit, disassemble, heat above 100°C (212°F), or incinerate.

Do not use if the package is open or damaged.

Do not touch the keyboard while inserting the battery.

## ADVERSE REACTIONS AND CONTRAINDICATIONS

It is the practitioner's full responsibility to read and follow those Instructions for use in order to avoid any adverse reactions or complaints.

Possible sides effects and contraindications can result from the use of products in conjunction with the DENTAPEN (e.g anesthesia cartridge, needle, etc.).

## HANDLING CONDITIONS

	OPERATING 0-2000 m	TRANSPORT & STORAGE
Temperature	20°C 68°F  25°C 77°F	-20°C -4°F  40°C 104°F
Humidity	60% RH AT 86°F / 30°C non-condensing	

It is recommended to store and transport DENTAPEN in its original packaging.

## CLEANING

Between patients, appropriate care should be taken to ensure the cleanness of DENTAPEN.

After each use clean DENTAPEN as soon as practical after use:

- Press the reverse button to bring the lead screw (free of any liquid) back to its original position.
- Wipe carefully DENTAPEN with a damp cloth to remove visible organic residue (e.g., residue of blood) and inorganic salts.

The finger grips and cartridge holders can be disassembled one from each other and autoclaved at 135°C/275°F for minimum 5 minutes. They can get a worn appearance after a number of autoclave cycles, so it is up to the Dentist to replace those regularly.

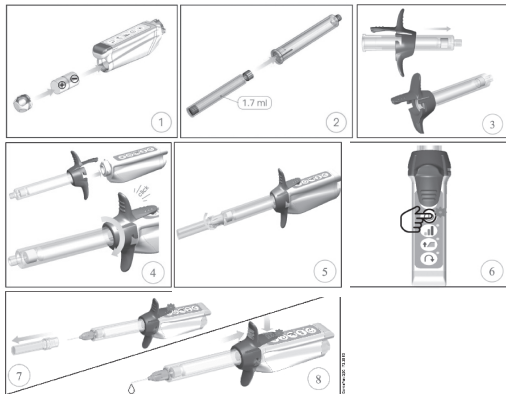
## Warning:

Do not immerse, soak or spray DENTAPEN, its screw and its accessories in or with liquids.


Do not use an automatic washer to clean the DENTAPEN.

Inspect DENTAPEN surfaces for breaks in integrity that would impair cleaning. Do not use DENTAPEN if it cannot be properly cleaned.»

## ASSEMBLY AND USE

1. Insert the battery.
2. Put the cartridge into the cartridge holder.
3. Engage the cartridge holder into the finger grips.
4. Slip DENTAPEN into a single-use sleeve, connect the finger grips to the main body and rotate until you hear a "Click".
5. Attach the needle onto the cartridge holder without removing the cap.
6. Press the ON/OFF button (automatic priming). The green LED indicates that the device is turned ON and ready to use. If the On/Off light is orange or red, change the battery.
7. Remove the needle cap and rotate the cartridge holder to orientate the needle as wished. Removing the needle cap as the power is off may spill anesthetic solution.
8. Select the delivery speed, ramp mode or intraligamentary mode.

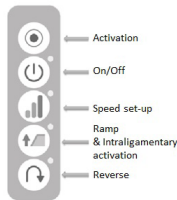ON/OFF button, light indicator

DENTAPEN inactivated:

- Green led: battery charged
- Orange led: weak battery
- Red led: battery to be replaced

DENTAPEN activated:

- Intraligamentary mode:
- Blinking blue led: low pressure
- Slow blinking purple led: expected pressure
- Fast blinking purple led: high pressure
- Other than intraligamentary modes:
- Slow blinking blue led: expected injection pressure

9. Press the top part of the finger grips to perform the injection. Pressing it another time stops the injection. The injection will automatically stop when the anesthesia cartridge is empty.

10. DENTAPEN turns off automatically, when not in use, after 4 minutes.

## Note:

In case of excessive pressure, DENTAPEN stops and displays an orange steady led. Reposition the needle and initiate again the injection.

To perform aspiration, the dentist should press the activation button twice in a short lapse of time. DENTAPEN will then generate a self-aspiration feature by pulling backwards its plunger and holding it still for a few seconds. This allows the Dentist to notice if blood gets into the cartridge. If so, the Dentists should remove the needle from this site and replace the cartridge by a new one to perform another injection.

If the O-ring is cracked or deteriorated, replace it prior to use: extend the plunger while no accessory is assembled, remove the old O-ring and slide a new one into the O-ring groove at the end of the plunger.

## WARRANTY

The warranty period of DENTAPEN is 24 months.

The warranty period for accessories is 3 months.

Battery excluded

## LEGEND

IP00	No protection against ingress of solid and liquid	Type BF applied part	Do not throw away
------	---	----------------------	-------------------

**⚠ WARNING:** This product can expose you to chemicals including bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproductive harm. For more information please refer to: <https://www.p65warnings.ca.gov/>

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EMC Information

DENTAPEN complies with the standard IEC 60601-1-2: 2014, Class B/Group 1 in emission.

Medical electrical equipment should be used with precautions according to EMC notices disclosed in this manual.

DENTAPEN is intended for use in the electromagnetic environment specified below. The customer or user of DENTAPEN should assure that it is used in such an environment.

Table 1 – Emission Requirements Comparison		
Emission test	Compliance	Electromagnetic environment - guidance
Conducted emissions CISPR 11: 2009 / AMD1: 2010	Not applicable	DENTAPEN uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. DENTAPEN is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Radiated emissions CISPR 11: 2009 / AMD1: 2010	Class B / Group 1	
Harmonic emissions IEC 61000-3-2:2018	Not applicable	
Voltage fluctuations flicker emissions IEC 61000-3-3: 2013/AMD1:2017	Not applicable	
Devices with motors or switching circuits CISPR 14-1	Not applicable	

Table 2 – Immunity Level Comparison – Transient Phenomenon			
Immunity test	IEC60601 test level	Verdict	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2: 2008	+/- 8 kV contact discharge +/- 15 kV air discharge	Pass	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient / burst IEC 61000-4-4: 2012	+/- 2 kV AC Mains +/- 1 kV I/O Ports 100 kHz PRR	Not applicable	
Surge – AC Mains IEC 61000-4-5: 2014 AMD1: 2017	+/- 2 kV (max) 0,90, 180 & 270 ° phase angles	Not applicable	
Surges – 12VDC Power ISO 7637-2	600 V	Not applicable	
Voltage dips & short interruptions immunity IEC 61000-4-11: 2004 AMD1: 2017	UT=0%, 0.5 cycle (0, 45, 90, 135, 180, 225, 270 & 315°) UT=0%; 1 cycle – UT=70% 25/30 cycles (0°) UT=0%; 250/300 cycles	Not applicable	
Conducted Immunity IEC 61000-4-6:2013	3V (0.15-80MHz) 6V (ISM bands)	Not Applicable	Mains power quality should be that of a typical commercial or hospital environment. DENTAPEN is a self-powered (non-rechargeable battery) handy device that is not connected to the public low-voltage power supply network.
Magnetic Immunity IEC 61000-4-8: 2009	30 A/m 50 and 60 Hz	Pass	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Table 4 – Immunity Level Comparison – Steady State Field Phenomenon		
Immunity test	IEC60601 test level	Verdict
Radiated RF IEC IEC 61000-4-3: 2006 AMD1:2007/AMD2:2010	10V/m 80 MHz to 2,7GHz	Pass
Proximity Field from Wireless Transmitters IEC 61000-4-3: 2006 AMD1:2007/AMD2:2010	9V/m to 28 V/m 15 specific frequencies	Pass
<p>Electromagnetic environment – guidance</p> <p>Portable and mobile RF communications equipment should be used no closer to any part of the DENTAPEN, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended Separation Distance <math>d = 1.17 \sqrt{P}</math> 0.15 MHz to 80 MHz - <math>d = 1.17 \sqrt{P}</math> 80 MHz to 800 MHz - <math>d = 2.33 \sqrt{P}</math> 800 MHz to 2,7GHz, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 		

Table 6 – Recommended separation distances between portable and mobile RF communications equipment and the DENTAPEN			
DENTAPEN is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the DENTAPEN can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and DENTAPEN – as recommended below, according to the maximum output power of the communications equipment.			
Rated Maximum Output Power of Transmitter (W)	Separation Distance According to Frequency of Transmitter (m)		
	150 kHz à 80 MHz $d = 1.17 \sqrt{P}$	80 MHz à 800 MHz $d = 1.17 \sqrt{P}$	800 MHz à 2,7GHz $d = 2.33 \sqrt{P}$
0.01	0.12 m	0.12 m	0.23 m
0.1	0.37 m	0.37 m	0.74 m
1	1.17 m	1.17 m	2.33 m
10	3.7 m	3.7 m	7.37 m
100	11.70 m	11.70 m	23.30 m

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